

# Analyser

28/07/2025 1:40 pm BST

## Summary

It is possible for Merlin to receive lab results directly from some of your practice's lab equipment. Our Customer Product Services team will provide guidance for your practice at the point of install.

Results retrieved from internal equipment is stored under the Analyser tab.


## Details

The Analyser tab can be accessed via Utilities > General Lab > Analyser.

## Using the Lab Analyser

### From Reception


1. From the Reception Screen, search for a client and patient.
2. Select the 'Lab' button.
3. The General Lab Screen will display with the Analyser tab selected.

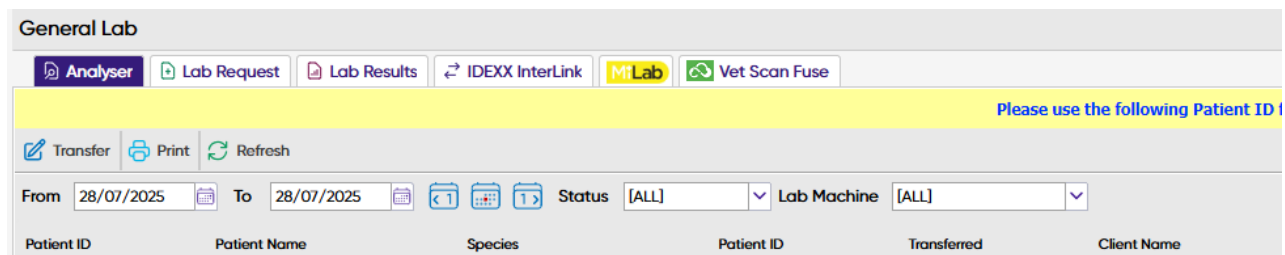

The screenshot shows the 'General Lab' interface. At the top, there's a header with the title 'General Lab' and a row of tabs: 'Analyser' (selected), 'Lab Request', 'Lab Results', 'IDEXX InterLink', 'MILab', and 'Vet Scan Fuse'. Below the tabs is a yellow banner with the text 'Please use the following Patient ID'. Underneath the banner are buttons for 'Transfer', 'Print', and 'Refresh'. Below these are filters for 'From' and 'To' dates (both set to 28/07/2025), 'Status' (set to [ALL]), and 'Lab Machine' (set to [ALL]). At the bottom, there's a table with columns: Patient ID, Patient Name, Species, Patient ID, Transferred, and Client Name.

### From Treatment

1. Navigate to a Patient's Treatment Screen.
2. Select the 'Patient Lab' button.

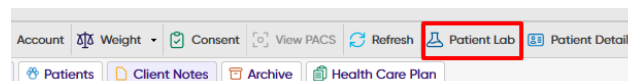

The screenshot shows a patient's treatment screen. At the top, there's a header with the title 'Patient Lab' and a row of buttons: 'Account', 'Weight', 'Consent', 'View PACS', 'Refresh', 'Patient Lab' (highlighted with a red box), and 'Patient Details'. Below the buttons are tabs for 'Patients', 'Client Notes', 'Archive', and 'Health Care Plan'.

3. The General Lab Screen will display with the Analyser tab selected.

## Requisition ID

To make a request on your lab machine, you must get a Requisition ID for the patient. This is displayed in

the large yellow box at the top of the analyser screen.

**Please use the following Patient ID for Finn: 1266570**

Follow the usual procedure for running your sample in your lab machine but when prompted for a Requisition ID, enter the Requisition ID for the patient you have noted.

## **View Analyser results**

When the tests have been run, the results will automatically be inserted into the patient's treatment history. You can view the results from the result line under the Analyser tab or from the Patient's treatment history.

---